ENG STETHOSCOPE LD. Instruction Manual.

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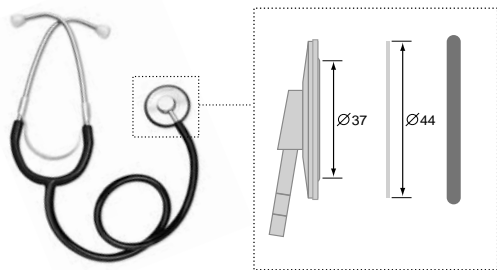
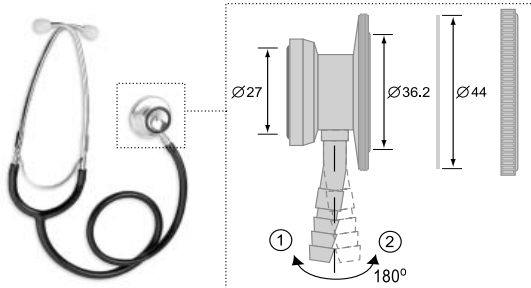
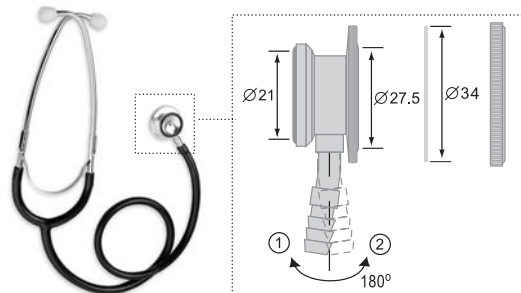
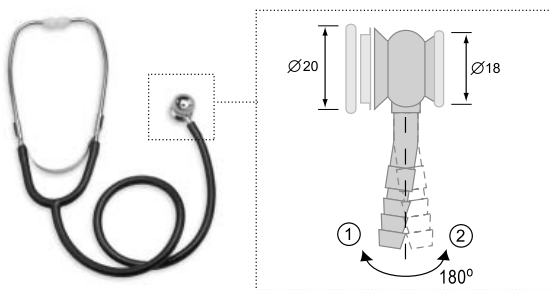
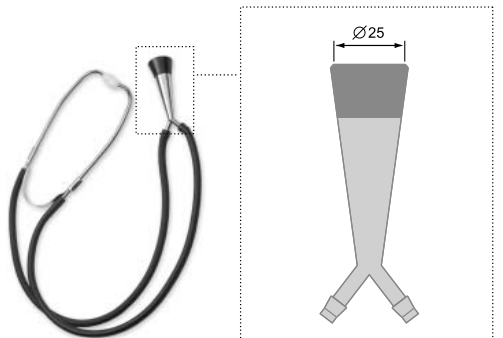
ENG

LD PROF-PLUS

LD Prof-Plus stethoscope is a Korotkov's tone phonendoscope consisting of a head that is shaped like a plastic diaphragm, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope head (LD-S025) with a 44 mm diaphragm (LD-S023) – 1 piece.
- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, grey, red, blue, purple, green, yellow) – 1 piece.
- Binaural tubes (LD-S019) – 1 piece.
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 76.

LD PROF-I

LD Prof-I stethoscope is a stethophonendoscope intended for medical and paramedical personnel that consists of a dual head with one side shaped like a plastic diaphragm and another side – like an open-end bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope dual head (LD-S017) with a large diaphragm and open-end bell – 1 piece. The diaphragm (LD-S023) has a diameter of 44 mm. The bell is fitted with a contact rubber ring. The diameter of the bell is 27 mm. Turn the head around its axial valve by 180 degrees to switch over between the bell and diaphragm. The bend of the axial head indicates to an active side of the head. To switch over to the diaphragm bring the bend to position (1) and to switch over to the bell bring the bend to position (2).
- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, grey, red, blue, purple, green, yellow) – 1 piece.
- Binaural tubes (LD-S019) – 1 piece.
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 135.

LD PROF-II

LD Prof-II stethoscope is a special pediatric stethophonendoscope consisting of a dual head with one side shaped like a plastic diaphragm and another side – like an open-end bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope dual head (LD-S028) with a small diaphragm and open-end bell – 1 piece. The diaphragm (LD-S024) has a diameter of 34 mm. The bell is fitted with a contact

rubber ring. The diameter of the bell is 21 mm. Turn the head around its axial valve by 180 degrees to switch over between the bell and diaphragm. The bend of the axial head indicates to an active side of the head. To switch over to the diaphragm bring the bend to position (1) and to switch over to the bell bring the bend to position (2).

- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, blue, green, yellow) – 1 piece;
- Binaural tubes (LD-S019) – 1 piece;
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 95.

LD PROF-III

LD Prof-III stethoscope is a special neonatal stethophonendoscope consisting of a dual head with one side shaped like a plastic diaphragm and another side – like an open-end bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope dual head (LD-S037) with a small diaphragm and bell – 1 piece. The diaphragm (LD-S040) has a diameter of 20 mm and bell – 18 mm. Turn the head around its axial valve by 180 degrees to switch over between the bell and diaphragm. The bend of the axial head indicates to an active side of the head. To switch over to the diaphragm bring the bend to position (1) and to switch over to the bell bring the bend to position (2).

- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, blue, green, yellow) – 1 piece;
- Binaural tubes (LD-S019) – 1 piece;
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 85.

LD PROF-IV

LD Prof-IV stethoscope is a special fetoscope consisting of a head that is shaped like a bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- Light-metal cone-shaped stethoscope head (LD-S038) with a plastic bell – 1 piece. The bell has a diameter of 25 mm.
- Medical connecting tubes (LD-S039) – two sound-conducting latex tubes of 56 cm length (optional colors: black, blue, purple, green) – 1 piece.
- Binaural tubes (LD-S019) – 1 piece.
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 115.

MAINTENANCE, STORAGE, SERVICE AND DISPOSAL

1. Protect the stethoscope from high humidity, direct sun exposure and shock loads.
2. Do not keep or use the stethoscope close to heaters and open fire.
3. Keep the stethoscope out of contamination.
4. Make sure that the stethoscope does not contact with any aggressive solutions.
5. When required, repair this device at authorized service centers only.
6. For disposal of the device follow the local regulation applicable at your region. No special disposal provisions are stipulated by the producer.
7. Operating conditions: +10°C to +40°C; relative humidity: 85% or below. Storage and transportation conditions: -34°C to +50°C; relative humidity: 85% or below.

WARRANTY

1. The following LD product is covered by warranty for the period specified in the warranty card.
2. The addresses of organizations providing warranty service can be found in section "MANUFACTURER AND AUTHORIZED REPRESENTATIVES" or on the website www.littledoctor.sg.

MANUFACTURER AND AUTHORIZED REPRESENTATIVES

SYMBOLS:

CE Compliance with Directive 93/42/EEC

Important: Read the manual

Representative in the European Union

Manufacturer

Protect from moisture

The revision date of this Instruction Manual is shown on the last page as EXXX/YYMM/NN, where YY is the year, MM is the month, and NN is the revision number.

This unit manufacturing is certified according to international standard ISO 13485. Stethoscope comply with the requirements of European Directive MDD 93/42/EEC.

Manufactured under control and for Little Doctor International (S) Pte. Ltd. (7500A BEACH ROAD #11-313 THE PLAZA SINGAPORE 199591. Postal address: Yishun Central P.O. Box 9293 Singapore 917699).

Manufacturer: Little Doctor Electronic (Nantong) Co. Ltd. (No. 8, Tongxing Road Economic & Technical Development Area, 226010 Nantong, Jiangsu, People's Republic of China).

Distributor in Europe: Little Doctor Europe Sp. z o.o., 57G Zawila Street, 30-390, Krakow, Poland, Sales Office phone: +48 12 2684746, 12 2684747, fax: +48 12 268 47 53. E-mail: biuro@littledoctor.pl www.LittleDoctor.pl

Authorized Representative in the EU: Little Doctor Europe Sp. z o.o., 57G Zawila Street Krakow 30-390 Poland.

For more information please visit www.littledoctor.sg

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